Hearing aid

Sound Control 16

medica+

JAPAN TECHNOLOGY

medica+

JAPAN TECHNOLOGY

Instruction Manual

Hearing aid

Sound Control 16

Content

Introduction	01
Overview	03
Working principle	05
Structure of hearing aid	06
Structure of charging case	07
Indicator light	09
Troubleshooting	15
Maintenance	17
Precautions	20
Unpacking and check	23
Contraindications	23
Warranty	24

Introduction

Dear Customer

Thank you for choosing our Hearing Aids. All illustrations in this booklet are applicable to our Hearing Aid. We are not only provides you with superior hearing aids and related products, but also provides you with comprehensive and considerate service. To thank you for your trust and support, we will wholeheartedly provide you with first-class after-sale service. You can get free repair of non-human causes of failure during the warranty period. Our customer service center consists of professional technical personnel and audiologist and you will find a satisfactory solution there.

This manual will introduce you some basic use methods of hearing aids and the solutions of common problems to help you use hearing aids better. Please read it carefully and regard it as a user guide.

Overview

This In-The-Ear Hearing Aid (Hearing Aid for below items) has been passed the strictly quality inspection to make sure the device in perfect performance.

a) Product Feature

The shape of the hearing aid conforms to the structure of the human ear canal and it is invisible and comfortable to wear. It is not only suitable for listening in quiet environments, but also for noisy environments.

b) Main uses and Application Scope

For hearing compensation for patients with hearing loss.

c) Product Model: Sound Control 16

d) Condition

Operating Conditions:

Temperature: $0^{\circ}\text{C} \sim +40^{\circ}\text{C}$,

Humidity: $0\% \text{ RH} \sim 80\% \text{ RH}$.

Rated Voltage: DC 3.7 V.

Storage and Transportation Condition:
Temperature: $-20^{\circ}\text{C} \sim +50^{\circ}\text{C}$,
Humidity: 0% RH \sim 93% RH.
Normal Atmospheric Pressure Range:
86 Kpa \sim 106 Kpa .

e) Impact on the Environment

Hearing aid contain a rechargeable polymer lithium battery. Please do not throw them away at will. The disposal of used battery must comply with local regulations.

f) Safety

- Warning: Keep small parts that can be swallowed out of children's reach.
- Warning: Do not put the hearing aid in a microwave oven or fire, as it may cause an explosion.
- The specific hearing aid must only be used by the intended person and not by others.
- Users who have undergone surgery

need to use hearing aid under the guidance of a professional doctor.

- Special User:

(1) Children can't wear the hearing aid without the guidance of adult.

(2) Mental patients should consult a physician before wearing a Sound Amplifier.

Working principle


Hearing aid is mainly composed of microphone, integrated circuit and receiver.

The Sound signal is converted into electrical one through the microphone and is magnified

via IC. Then the electrical signal is converted into the audible signal through the receiver.


Structure of hearing aid

- 1 – Charging port
- 2 – Shell
- 3 – Earplug
- 4 – Microphone
- 5 – Button
- 6 – Pull wire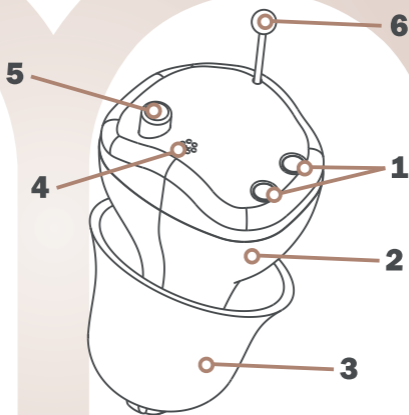

Structure of charging case

- 1 – Charging Indicator of Charging Case
- 2 – Type-C Port to External Power Supply
- 3 – Charging Indicator of Hearing Aids
- 4 – Hearing Aid Charging Insert
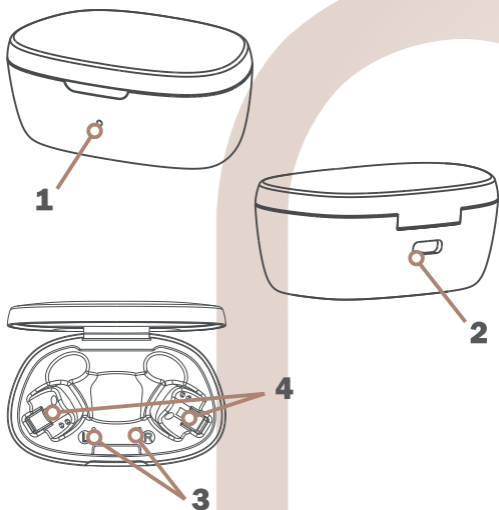


Structure of charging case

a) Power on/off

1. Power On: Take the hearing aid away from the charging case and it will automatically turn on. Power on delay 5~6 seconds, the purpose is to prevent feedback noise.

2. Power Off: The hearing aid is automatically turned off when it is placed in the charging case.

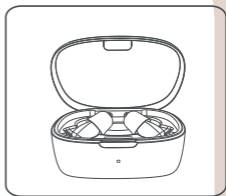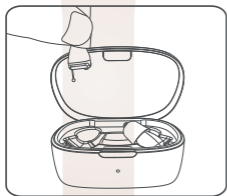Charging condition:


It takes 3~4 hours to fully charge the charging box; 2-3 hours to fully charge

the hearing aid;
 when the charging box is fully charged, it
 can charge both hearing aids 3 times at
 the same time;
 when the hearing aid is fully charged, it
 can be used for 25~30 hours.

Indicator light

Charging Indicator (case in charging)		
Working Condition	Indicator	Status
Without hearing aid in case		Case in charging, lights on
		Case fully charged, lights off
With hearing aid in case		Hearing aids in charging, lights on
		Hearing aids fully charged, lights off
		Hearing aids and case fully charged, lights off

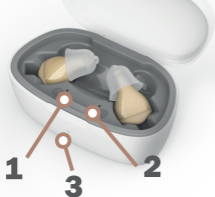Charging Indicator (case not in charging)



Working Condition	Indicator	Status
Without hearing aid in case		Case on standby
With hearing aid in case		Hearing aids or Case, no charge, no power
		Hearing aids in charging, lights on
		Hearing aids fully charged, lights off

Note: If the battery capacity of the hearing aid exceeds 90%, it will not be recharged if it is placed in the charging box (the purpose is to prolong the battery life).


b) Replacing Earplug

1. Remove the earplug: Pinch the earplug tightly with your thumb, index finger, and middle finger to separate the earplug from the connector.
2. Install earplug: Choose the right earplug according to the size of the ear canal, pinch the earplug into the hearing aid connector, and install them in place.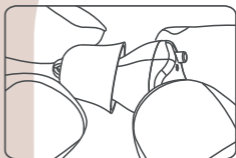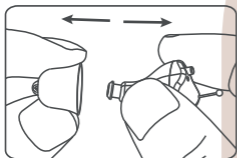



c) Wearing Hearing Aids

Hold the hearing aids with your thumb and forefinger, with the pull wire underneath. Use the tip of your index finger to gently push the hearing aid into your ear until it is comfortable in the ear canal. Pulling the outer ear back and up with the other hand will make it easier to put the hearing aid on. Use your index finger to move up and down gently, open and close your mouth several times to check whether the user is wearing it well. (The button is exposed outside after wearing)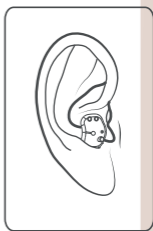

d) Removing Hearing Aids

Pinch the pull wire with your thumb and forefinger and gently pull it out of your ear canal.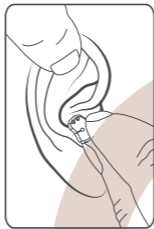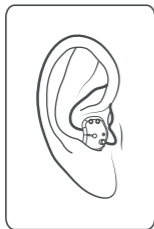


e) Volume Control

Volume adjustment: Short press the button to adjust the volume circularly. When you short press, you will hear two "beeps", the volume will become louder, you will hear three "beeps", the volume will be the highest, and you will hear three "beeps" again, the volume is the smallest, and followed by recycling.

Soft power off or power on: Long press the button for 3 seconds to soft power off or power on.

Digital button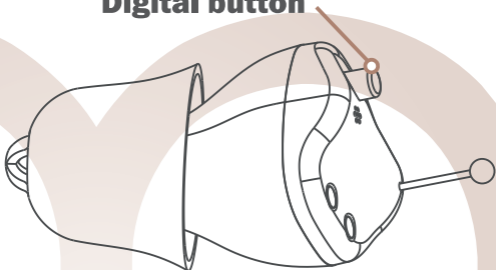

Troubleshooting

Whistling

1. Hearing aid is not worn correctly. Re-wear correctly.
2. Too much earwax. Clean earwax.

Silence

1. Low battery or no power. Charge the hearing aid.
2. Dirt blocks the sound tube or earplug. Clean hearing aids.
3. Dirt blocks the earwax guard shield. Replace earwax guard.

Small volume

1. Too low volume. Turn up the volume.
2. Low battery. Charge the hearing aids.
3. Hearing aid is damped. Use the dryer.
4. Dirt blocks the sound tube or earplug. Clean hearing aids.
5. Dirt blocks the earwax guard shield. Replace earwax guard.

High power consumption

1. Hearing aid is still on when not in use. Please place it in the charging case when hearing aid is not in use.
2. Hearing aid is damped. Use the dryer.

Intermittent Sound

1. Low battery. Charge the hearing aids.
2. Dirt blocks the sound tube or earplug. Clean hearing aids.
3. Dirt blocks the earwax guard shield. Replace earwax guard.

If the failure still not be cleared with the above methods, please contact the hearing center where you bought the hearing aid or contact with customer service center. Don't open or repair the device by yourself.



Maintenance


During the use of earwax will accumulate in the ear canal and the sound hold of the hearing aid. Too much earwax will affect the use of hearing aids. So please clean your ear canal and maintain hearing aids regularly.

a) Clean Earplug



Remove the earplugs of the hearing aids, use a cleaning cloth to clean the earwax or use water to wash, then dry the earplugs, make sure no water remaining.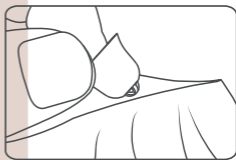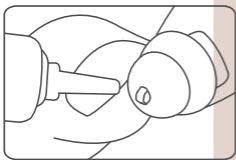

b) Replace Wax Guard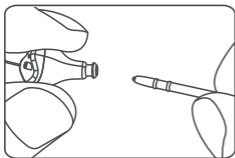


Please follow the instructions on the Wax Guard package to replace the Wax Guard.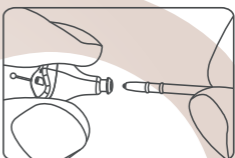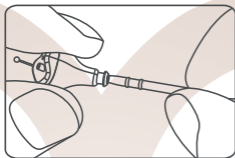

c) Clean Hearing Aids

Clean the hearing aids on a soft table to avoid damaging and dropping.

Don't use any liquid cleaner to clean it since they may lead to permanent damage to the circuitry.

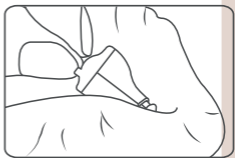d) Moisture Prevention

Please put the hearing aids into the drying box before bedtime every night to prolong the life of the hearing aids. Note: Dryer are sold separately.

e) Vibration Prevention

Put the hearing aids in a charging case when you don't wear the hearing aids.

Precautions

a) Moisture Prevention

Any form of moisture has an impact on hearing aids. Moisture prevention can help prolong the life of hearing aids. Avoid using or storing them in humid environment and dry hearing aid regularly.

b) Water Prevention

Keep hearing aids away from water. Please take off the hearing aids in advance before swimming, shower, washing the head or face. If hearing aids falls into the water accidentally, do not use any oven or similar dryer to dry it. You can use the soft cloth to dry it and put it in a ventilated and dry environment. If any failure occurs, please contact the hearing center where

bought hearing aids or customer service center.

c) High Temperature Prevention

Temperatures or prolonged exposure to sunlight.

d) Dropping Prevention

Hearing aids are precise electronic devices, should avoid falling to the hard ground or violent vibrations and shocks.

e) EMC

According to IEC 60118-13, please take off your hearing aids before you get close to radio and telecommunication equipment, otherwise that could damage your hearing or hearing aids.

Portable and mobile radio frequency communication devices may affect the normal use of hearing aids. Use hearing aids in the recommended electromagnetic environment.

f) Others

- Do not wear hearing aids during X-ray, MR scans, CT scans, short-wave diathermy, or similar radiation treatment.
- Hearing aids should be regularly sent to the fitting center for maintenance and inspection.

Unpacking and check



- Check whether the model marked on the warranty card is consistent with the hearing aids model.
- Check whether the serial number marked on the warranty card is consistent with hearing aids serial number.

Contraindications

- Acute otitis externa, otitis media, chronic suppurative otitis media (in the period of purulent infection), acute suppurative otitis media, patients allergic to the materials of this product. The selected hearing aids should undergo professional hearing testing and fitting, and be used under the professional guidance of a doctor or hearing aid audiologist.

Warranty card for service records

The product warranty is provided by the manufacturer for medica+ products from 12-36 months.

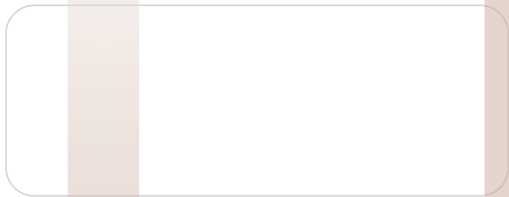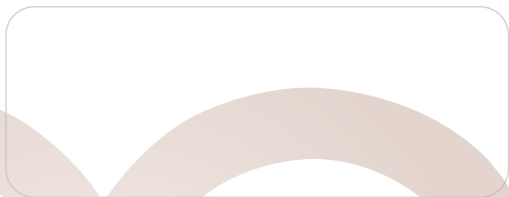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