ENG / UA / PL

Aspirator

Nose Cleaner 6.0

medica+
JAPAN TECHNOLOGY

medica+
JAPAN TECHNOLOGY

ENG

Aspirator
Nose Cleaner 6.0

User Manual


Contents

Overview of the Aspirator.....	01
Safety Instructions	02
Usage Instructions	03-04
Cleaning	05-06
Technical Specifications	07
Package Contents	07
Warranty Obligations.....	08

Dear Customer!
Thank you for choosing Medica+ products.
Please read this manual carefully before use.

Overview of the Aspirator

The Medica+ NOSE CLEANER 6.0 is an electric nasal aspirator for children and adults, designed for effective mucus removal from nasal passages. With multiple power levels, the device is suitable for all ages, from newborns to adults.

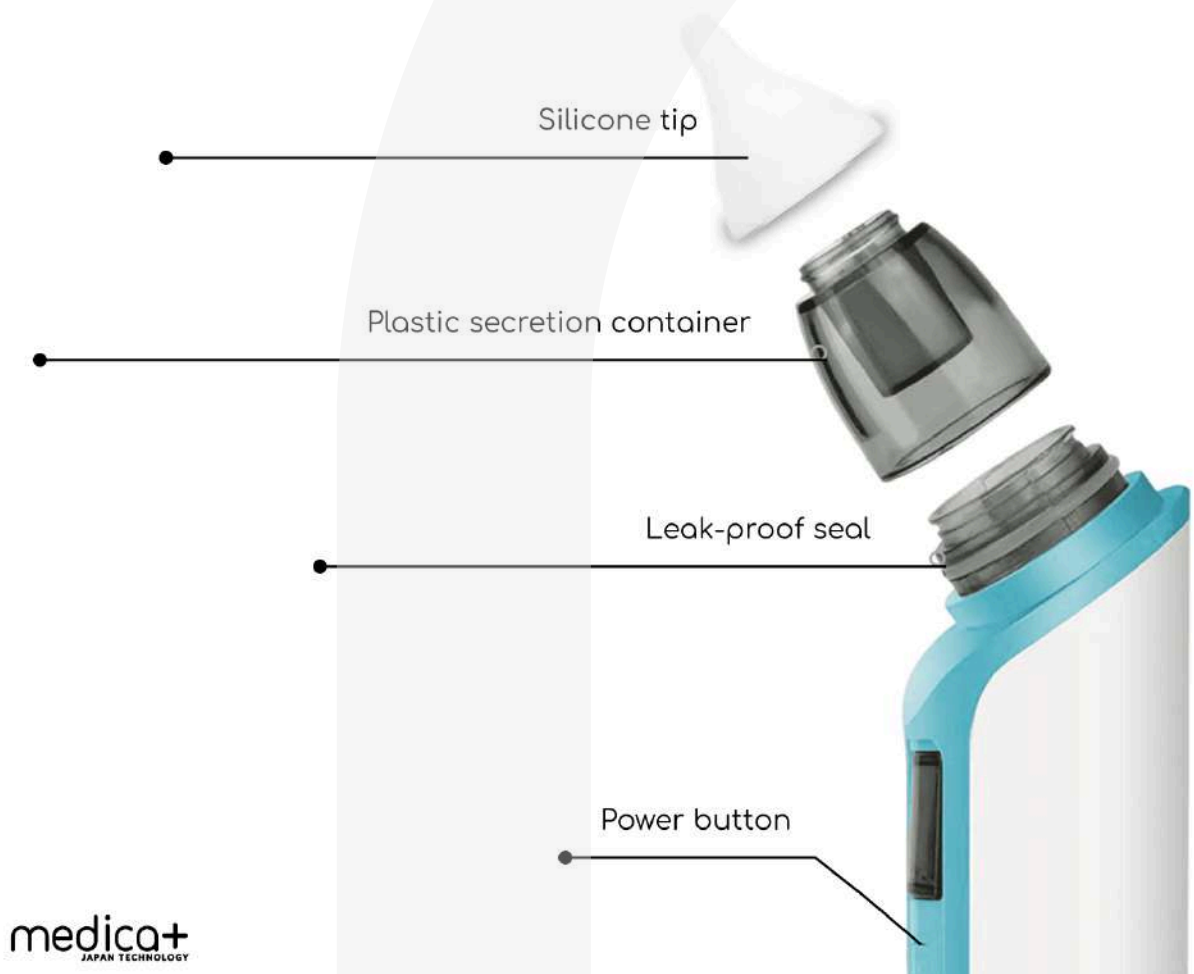Safety Instructions

Warning!

- Before use, ensure the aspirator is properly assembled.
- If the device starts malfunctioning, turn it off immediately.
- Keep the aspirator out of reach of children.
- Do not use or store the device near heat sources, hot surfaces, humid, or dusty environments.
- Do not store the aspirator with chemicals or flammable gases.
- If the device is damaged, contact a service center.
- Do not modify the device settings.
- Regularly inspect the aspirator and its components before use.
- Avoid prolonged use of the device.
- Do not insert the aspirator too deep into the nasal passages.
- Do not use the device for cleaning ears, mouth, or eyes.
- The aspirator is intended solely for nasal secretion removal in a home setting. Improper use may be hazardous.
- Use only the USB cable provided in the package for charging.

Usage Instructions

Using the Aspirator

1. Select one of the six power levels by pressing the suction power button. Adjust the power according to your needs.

6 POWER LEVELS

For mild, clear mucus secretion

For liquid secretion

For moderate secretion

For heavy secretion

For thick, sticky secretion

For very thick secretion

medica+
JAPAN TECHNOLOGY

1. Lay the child on their back or hold their head steady during use.
2. Insert the nozzle gently, avoiding deep insertion into the nose.

How to Effectively Remove Mucus

1. Place the nozzle in the nostril and make gentle circular motions to help clear any remaining mucus.
2. If there are dry nasal secretions, use a saline solution:
 - Spray the solution into the nostril.
 - Wait 3–4 minutes for the secretions to soften.
 - Remove them with the aspirator.
3. For better suction, press the nozzle closer to the nasal cavity. This creates a vacuum, making mucus removal easier.

Monitoring the Reservoir Fill Level

1. Keep an eye on the reservoir fill level during use.
2. If it becomes full, secretions may enter the suction port and cause malfunctions.
3. If the reservoir is overflowing, turn off the device immediately and clean it.

Cleaning


How to Clean the Reservoir

1. Turn off the aspirator and detach the reservoir from the device.
2. Carefully pour out the contents of the reservoir.
3. Wash the reservoir with warm water and a mild detergent.
4. Rinse thoroughly with clean water and dry completely before the next use.
5. Ensure the reservoir is fully dry before reconnecting it to the device.

How to Sterilize the Nozzles

1. Remove the nozzles from the device.
2. Wash them with warm water and a mild detergent.
3. To sterilize, place the nozzles in boiling water for 3–5 minutes or use a steam sterilizer.
4. Allow the nozzles to dry completely before reuse.

Regular cleaning and sterilization help maintain hygiene and ensure the aspirator's effective performance.

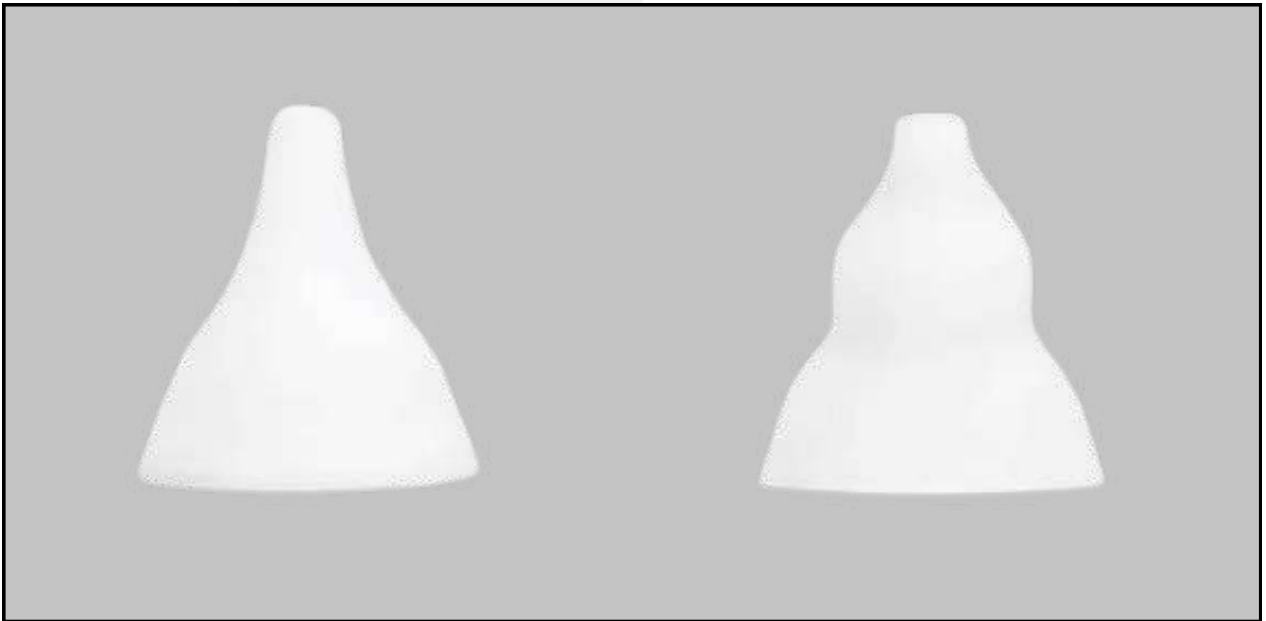Technical Specifications

1. Battery – 500mAh
2. Power – 5W
3. Voltage – 3.7V

Package Contents

Aspirator – 1 pc.
Hypoallergenic nozzles – 2 pcs.
USB charging cable – 1 pc.
User manual – 1 pc.

Warranty Terms

Dear Customers!

Please keep this manual properly to use the warranty certificate. The warranty is valid for one year from the date of purchase (or two to three years if an extended warranty is purchased) in case of malfunction under normal operating conditions.

The warranty does not cover:

1. Damage caused by improper use or lack of proper maintenance.
2. Damage resulting from disassembly or modifications.
3. Use of non-original parts or accessories not supplied by our company.
4. Damage caused by force majeure events.

According to the warranty terms, any out-of-warranty maintenance is subject to additional charges.

Parts replaced during the warranty period remain the property of MEDICA+.

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