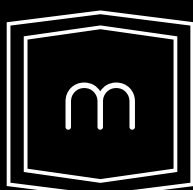
ENG / UA / PL

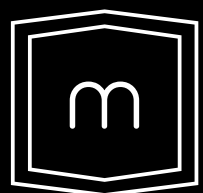
Hearing aid

Sound Control 15


medica+
JAPAN TECHNOLOGY

medica+

JAPAN TECHNOLOGY

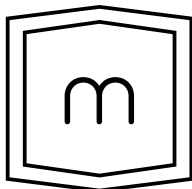ENG

medica+
JAPAN TECHNOLOGY

Instruction manual

Sound Control 15

Content

Operating Principles.....	01
Features.....	02
Installation Steps.....	03
Turn on/off.....	04
Volume Control.....	05
Mode selection.....	05-06
Applying your Hearing Amplifiers	06-07
Removing your Hearing Amplifiers.....	08
Charger Case.....	09
Functional Operation.....	09
Charging Hearing Amplifiers.....	10-11
General Precautions.....	12
Warranty Conditions.....	13
Technical Specifications.....	14

Dear Customer!

Thank you for choosing Medica+ products. Please read this manual carefully before using the device.

Operating Principles

The hearing aid captures sound through a microphone and then converts it into an electrical signal. This signal is amplified, making sounds clearer and louder for the user.

The level of amplification may vary depending on the degree of hearing loss. It can be adjusted in two ways:

- Pre-configured – by an audiologist during the selection process.
- Manually – using a button or device settings.

This ensures a comfortable sound experience tailored to the user's individual needs.

Features


1	Ear Tube
2	Mode/Power on button
3	Earbud
4	Volume +/-
5	Silicone Tube
6	Charging port

Installation Steps

1. Select an earbud of the appropriate size for comfortable wearing.

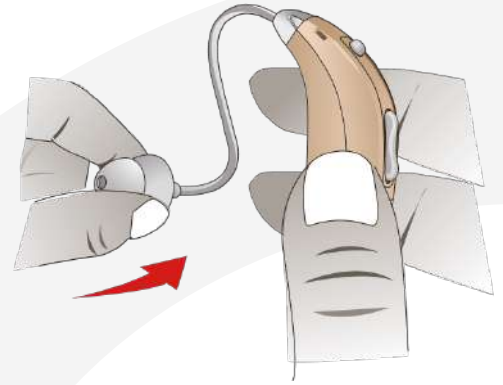2. Attach the earbud to the ear tube.

Turning your instrument on and off

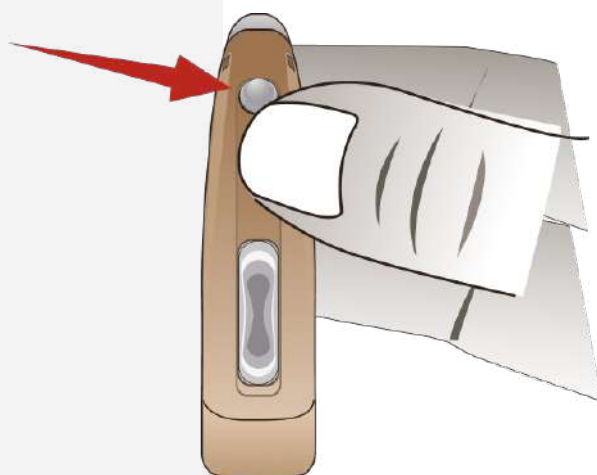
Turning On

The hearing aid amplifies sounds, making them clearer for the user. The level of amplification depends on the degree of hearing loss and can be preset by an audiologist or adjusted manually using the buttons.

Press and hold the power button for 3 seconds to turn on the device.


Turning Off

Press and hold the power button for 3 seconds to turn off the device. The indicator will blink twice, and a sound signal will play before the device shuts down.

Volume Control

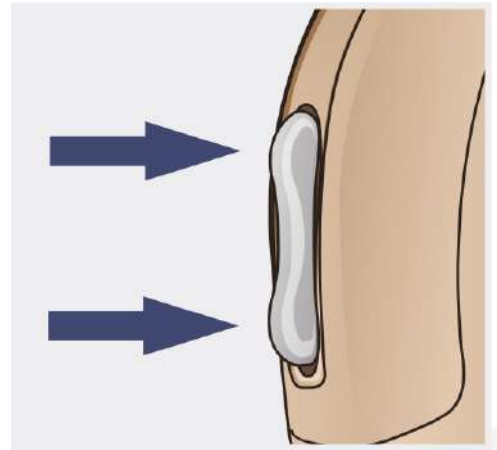- Press up to increase the volume.
- Press down to decrease the volume.


Mode selection

A short press of the power button allows you to select the desired mode.

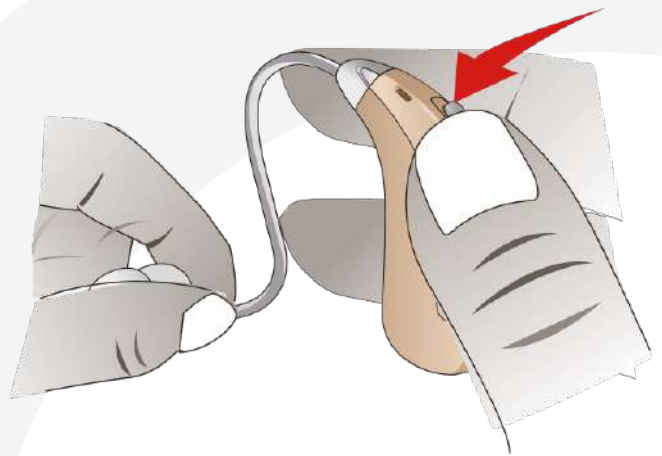Your hearing aid has three pre-set unique modes.


Switch between them with a short press – a sound signal will indicate which program is currently active. The hearing aid remembers the selected mode and automatically saves it when turned on or off.


Normal Mode		1 beep
Noise reduction mode		2 beep
Outdoor mode		3 beep

Applying your Hearing Amplifiers

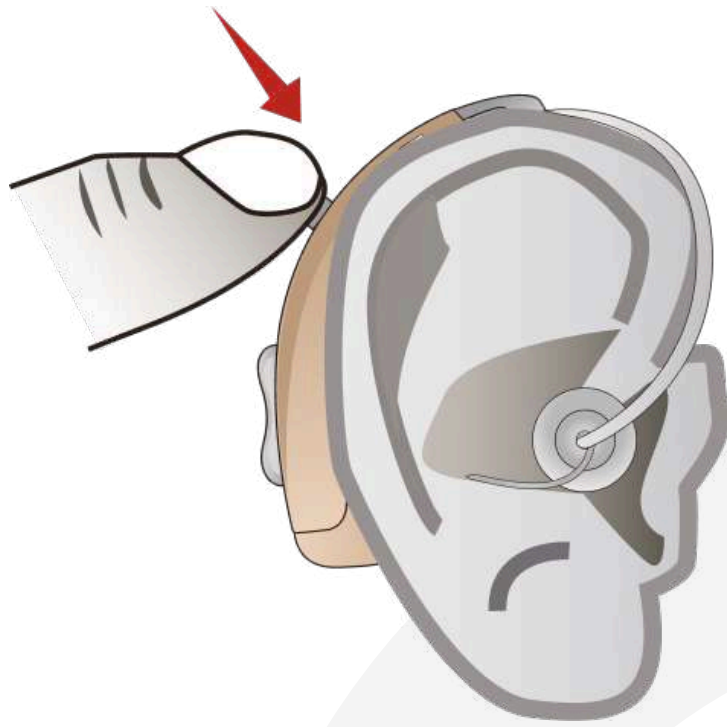1. Hold the ear tube between your index finger and thumb, then carefully insert it into the ear canal.
2. Gently pull the earlobe backward to ease the insertion of the earbud.
3. Note: Before inserting the hearing aid, turn it off or lower the volume. Try to insert the earbud as deeply as possible to prevent whistling sounds.
4. Secure the hearing aid behind the ear, ensuring the silicone earbud fits properly in the ear cavity.
5. Press and hold the power/mode button to activate the hearing aid.
 - (R1) Adjust the volume using the buttons to reach a comfortable sound level.
 - (R2) Ensure the sound is clear and pleasant for use.

P1

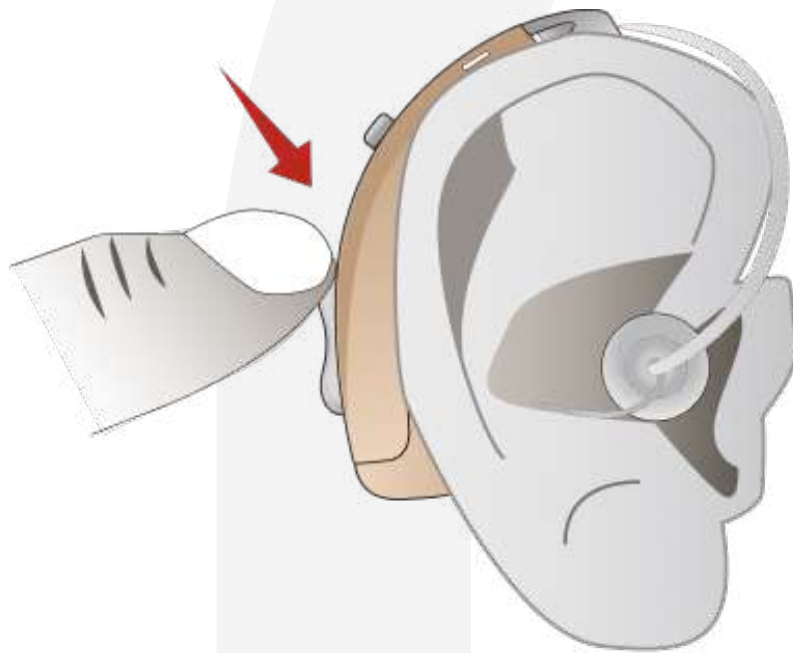
P2


Removing your Hearing Amplifiers

Hold the power button until the device is fully turned off. (P1)

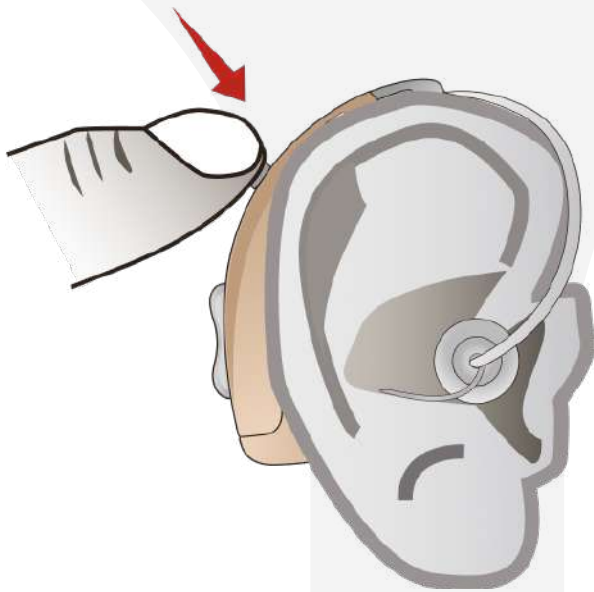1. Pull the silicone tube gently.
 2. Carefully remove the device from the ear.
- (P2)

P1

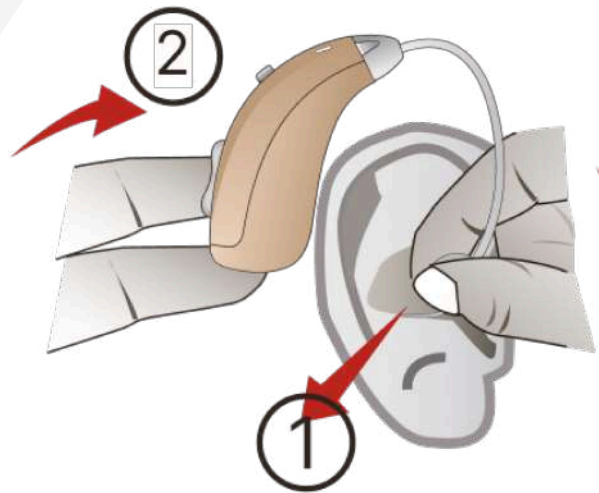P2

Charger Case

- | | |
|---|-------------------|
| 1 | Light Indicator |
| 2 | Charging Socket |
| 3 | USB Charging Port |

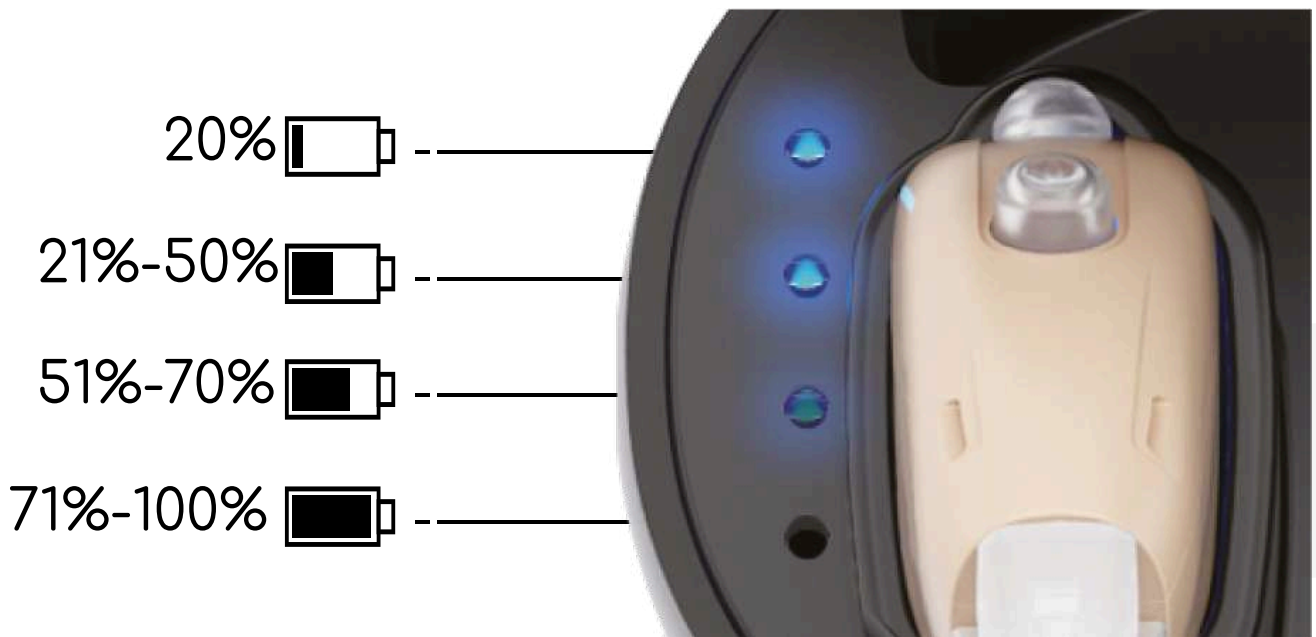
Functional Operation

When using the hearing aid for the first time (or after a long break), be sure to fully charge both the device and the case before use.

Charging Hearing Amplifiers

1. Open the charging case lid and place the hearing aid into the corresponding slot. The indicator will begin to flash slowly in blue, indicating that charging has started..
2. During charging, the blue indicators on the charging case will flash slowly, indicating the charging process and the battery level.

3. When the hearing aid is fully charged, the indicator will glow continuously.

General Precautions

1. Keep the device clean and dry. After use, wipe the body with a soft cloth or wipe to remove dust, dirt, and moisture.
2. Do not immerse the hearing aid in water or any other liquids, as this may cause irreversible damage.
3. Avoid drops and impacts. Do not throw the device on hard surfaces to prevent breakage.
4. Do not leave the hearing aid under direct sunlight or in places with high temperatures, such as a hot car. Excessive heat can cause the body to deform.
5. Do not use the hearing aid while showering, swimming, in heavy rain, or in humid environments like saunas or steam rooms.

Warranty Conditions

Dear customers,
Please keep this manual properly to use your warranty card. The warranty is provided for one year from the date of purchase (or for 2 or 3 years if an extended warranty is purchased) in case of malfunction under normal operating conditions.

The warranty does not cover the following cases:

1. If the product is damaged due to improper use or lack of proper maintenance.
2. If the product is damaged due to disassembly or modification.
3. If parts not belonging to our company were applied to and used with the device.
4. Damage caused by force majeure circumstances.

According to the conditions, technical services provided outside the warranty period are subject to additional charges.

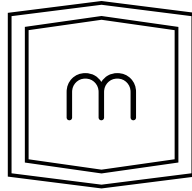
Parts replaced during the warranty period remain the property of MEDICA+.

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